



GI RESTORATIVE

SECURE GI FILL LC

THE COMMAND SET RESIN REINFORCED GI

Treating pediatric, geriatric & restricted mouth opening patients often poses unique challenges for clinicians. High levels of patient anxiety and limited cooperation often tend to restrict available chair time.

SELECT GI FILL LC reduces the haste hustle experienced while treating such patients. Its unique non-sticky, packable consistency makes it quick and easy to place. And specially incorporated dual cure feature helps limit the risk of salivary contamination; thereby achieving long lasting durable restorations.





SECURE GI FILL LC offers all the benefits of high strength GI restorative with the added advantage of resin reinforcement and command set dual cure; thus delivering higher strength, better integrity & longevity.

Non-sticky and hassle-free handling with superior compressive strength and packability

Command set cure allows sufficient working time and helps prevent saliva contamination

Dual cure feature with 3mm depth of light cure ensures complete curing

Extra translucency provides enhanced aesthetics

Special **bridge shade** helps achieve superior shade match with minimal inventory

Fluoride releasing anti-cariogenic properties

Resin reinforced SELECT GI FILL LC, helps limit the dehydration of GIC in mouth breathers and xerostomia patients, thus increasing the longevity of the restoration.



SPECIALLY INDICATED FOR

Class I *

Class III

Core Build Up



Class II *

Class V

*Non stress bearing

USAGE

- ⌚ Mixing Time 30 secs
- ⌚ Working Time** 2 min (minimum)
- ⌚ Self Cure Setting Time*** 6 mins
- ⌚ Light Cure 20 secs (minimum)

Working Time is at 23 ± 1 °C, *Self Cure Setting is at 37 ± 1 °C

ORDERING INFORMATION

- 📦 15 gm powder
- 📦 6 ml liquid
- 📦 1 shade – A2 ••• A3

[Click here](#) to find more on what other dentists are talking about Wizdent Products

📞 7506698711

✉ wizdent@pidilite.com

📘 fb.com/fineartofdentistry

🌐 www.wizdent.com